

This product conforms to UL and CSA standards



Solutions Start Here www.smpcanada.com

SMP Canada

10 Sims Crescent, Richmond Hill Ontario, L4B-1K9 Canada Tel: (905) 764-7767 Fax: 1-866-627-9798 support@smpcanada.com





INSTALLATION INSTRUCTION

www.smpcanada.com | support@smpcanada.com | Tel: 1.905.764.7SMP (1.905.764.7767) | Fax: 1.866.627.9798 10 Sims Crescent, Richmond Hill Ontario, L4B 1K9, Canada



INSTALLATION INSTRUCTION

BED DOCKING STATION BED DOCKER

Installation of a bed docker unit

- 1 Remove faceplates from the bed docker unit.
- 2 Place bed docker assembly against wall at desired location.



3 Jack or support on blocks to the desired height.



- 4 Locate pre-drilled mounting holes along the inside back plate of unit.
- 5 Mark wall through holes remove unit.
- 6 Drill holes for screws.



7 Place unit back against the wall, screw bed docker to wall through provided mounting holes.



INSTALLATION INSTRUCTION

BED DOCKING STATION BED DOCKER

Electrical connections

- 1 Feed wires from the back of the wall through conduit attached to termination box and attach to corresponding terminal block.
- 2 Close electrical section of the unit with stainless steel cover plate.



- 3 Test all electrical circuits.
- 4 Using supplied rubber suction cup, install plastic laminate by inserting top of the panel into upper channel, position panel vertically, then allow bottom edge to fall into lower channel.



5 Secure plastic laminate cover with the screws.

