


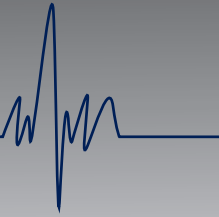

Solutions Start Here...



Solutions Start Here


www.smpcanada.com

SMP Canada

10 Sims Crescent, Richmond Hill
Ontario, L4B-1K9 Canada

Tel: (905) 764-7767 Fax: 1-866-627-9798
support@smpcanada.com

SMP 385MT

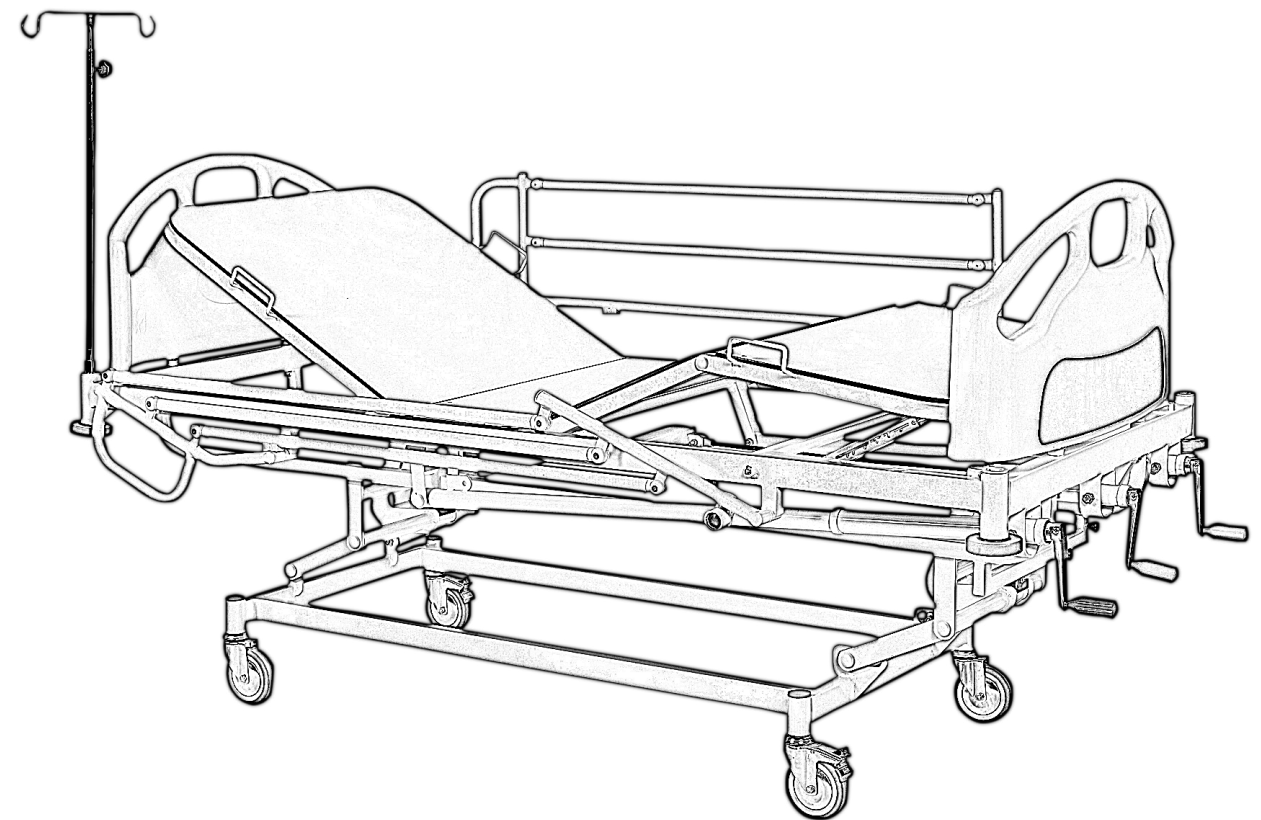

USER & MAINTENANCE MANUAL




USER MANUAL

MANUAL BED SMP 385MT

APPENDIX C SMP-385MT

Manual Bed – User & maintenance Manual

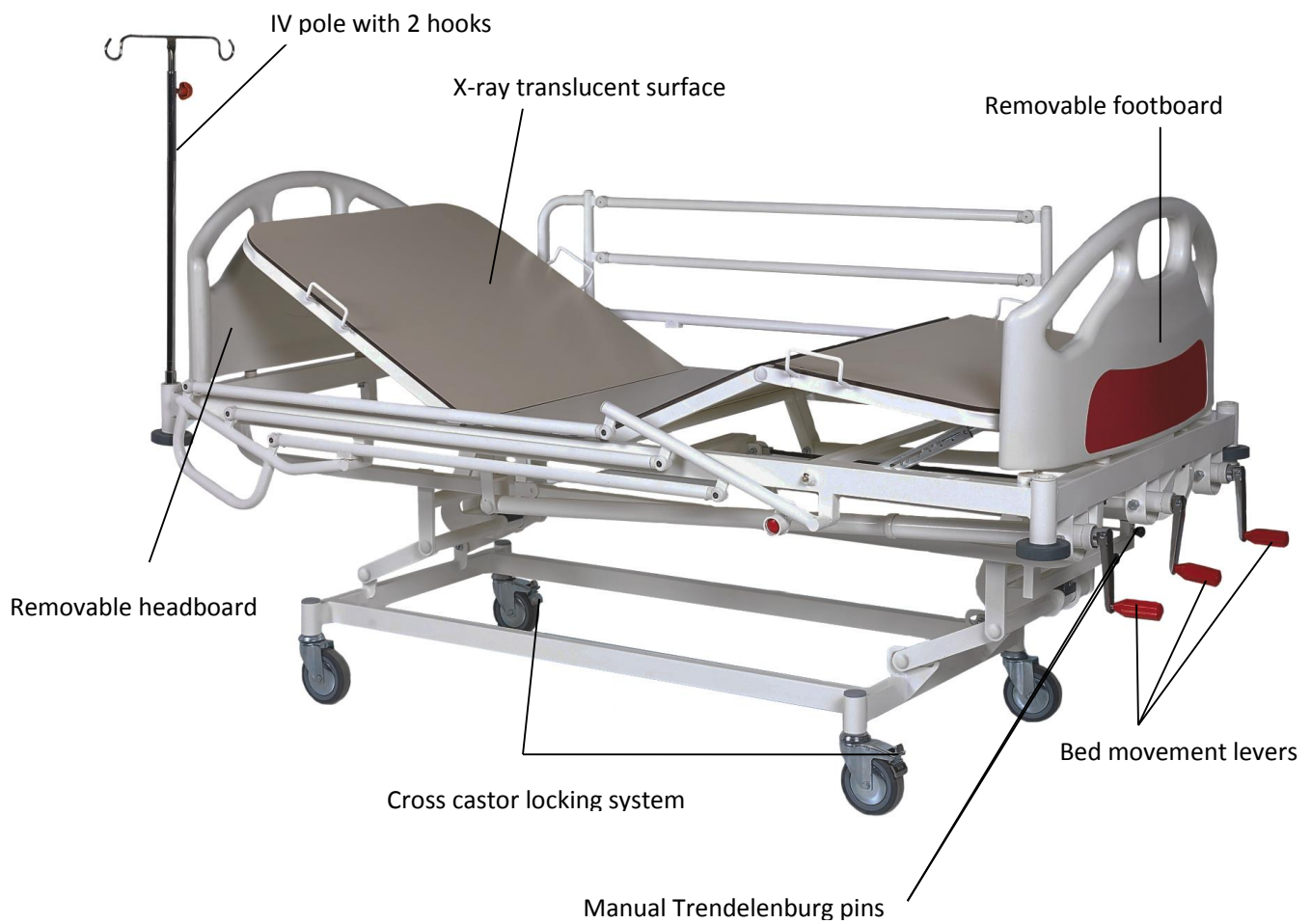



USER MANUAL

MANUAL BED **SMP 385MT**

General Remarks

We congratulate you on your choice of a SMP-385MT manual bed and would like to thank you for your confidence in our company and its products. You have purchased a SMP-385MT manual bed, developed and manufactured in accordance with the current state of technical knowledge.

SMP Canada's general patient room beds fully meet all safety and operational requirements. They are tested in accordance with international standard and carry the CE seal. This confirms that they fulfill the safety requirements relating to medical products. This user manual is intended to give you and your staff practical information and tips on the safe and correct use of the SMP-385MT manual bed.

When putting the bed into service for the first time, or wanting to learn about its operation and maintenance, one must read and strictly obey this user manual. To avoid faulty handling and to ensure trouble-free operation of the bed, this documentation must be available to caregivers and/or patients at all times.

Secondary Medical Products Canada Inc.

SMP Canada

10 Sims Crescent

Richmond Hill, Ontario, L4B 1K9 Canada

Tel.: ++1 905-764-7767

Fax: ++1 866-627-9798

E-Mail: info@smpcanada.com

Website: www.smpcanada.com



USER MANUAL

MANUAL BED SMP 385MT

Table of Contents

APPENDIX C.....	1
General Remarks.....	2
1. Safety Instructions	4
2. Bed Installation	4
3. Bed Usage	5
3.1 Side Rails	5
3.2 Cross castor locking system	5
3.3 Legrest adjustment mechanism.....	6
3.4 Mechanical Trendelenburg/Reverse Trendelenburg.....	6
3.5 Bed Movement Levers.....	6
3.6 Head/Foot Boards.....	6
4. Bed Cleaning	7
5. Storage.....	7
6. Maintenance Inspection Schedule	7



USER MANUAL

MANUAL BED SMP 385MT

1. Safety Instructions

- Read the statements with detail and use this product in accordance with these statements. If necessary, place a copy of these instructions near the product.
- If you notice the product is not working, do not use it and contact SMP's technical service right away.
- Inform the patient and the other users about the functions of the product.
- Pay attention the situation of the patient and around before operating the patient bed.
- The safety weight capacity should not be exceeded. If there is an obligatory situation to exceed this limit, the surface which the patient lies on should be adjusted to lowest position and the functions of the bed should not be used.
- When the patient is on the bed, a second person should not sit on the bed.
- Before cleaning the bed or changing any part of the bed, always pull the power cable out the plug.
- The patient and the attendant are always in danger in the following situations:
 - In the course of taking the bed away from one place to another on crinkled or grungy surface
 - In the course of exceeding the safe working weight
 - In the course of placing the moving parts carelessly
 - In the course of being damaged of the power cable
- Only the approved brands' parts should be used. When the replacement parts which are provided from the other suppliers are used, the bed manufacturer company will not accept responsibility for any damage, loss or injury.

2. Bed Installation

During the installation of the bed, please obey the following rules:

- Open the package of the bed completely and carefully
- Check the bed for the possible damages during the carriage
- Check the accessories and the main parts of the bed for any deficiency carefully, in case of any deficiency, contact the service department
- Before taking action about the bed, read the user's manual carefully

WARNING

DO NOT PUT ANYTHING ON THE BOTTOM FRAME COVER



USER MANUAL


MANUAL BED SMP 385MT

3. Bed Usage

3.1 Side Rails

There are two side rails on the bed for the safety of the patients. The side rails provide the lying patient not to fall and to avoid from external factors. The safe carriage capacity of each side rail is 75 kg.

To close the side rail, pull the red handle and let the side rail down. To open the side rail, pull the closed side rail upward. In both movements, the side rail locking mechanism becomes active automatically and a click noise assures locking of the side rail in place. The side rails can be opened/closed by the help of the red button located on the foot end of the side rails.

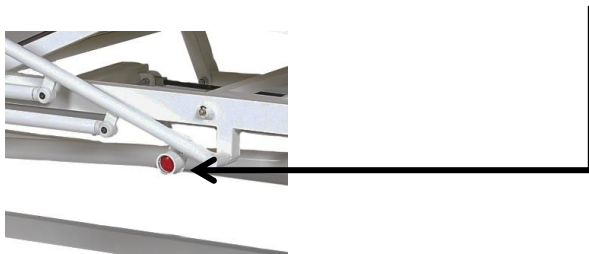

WARNING

MAKE SURE THERE IS NOTHING AND NOBODY, ESPECIALLY CHILDREN, UNDER AND NEAR THE BED WHEN OPERATING THE FUNCTIONS OF THE BED.

3.2 Cross castor locking system

To activate the lock, simply push down on the castor brakes. There is one castor brake located on each end of the bed. To unlock the castor brakes simply lift the lock as per diagram. Please make sure to unlock both castor brakes before moving the bed. Releasing the central brake will allow the bed to move freely.



USER MANUAL

MANUAL BED SMP 385MT

3.3 Legrest adjustment mechanism

Legrest movement is done by the help of a graded mechanism. Upholding the legrest is enough to uphold the graded mechanism. To lower again, uphold the legrest and then let it down. The legrest will come to straight position



3.4 Mechanical Trendelenburg/Reverse Trendelenburg

The bed can be put into Trendelenburg positions by pulling on the pins located at the foot end of the bed. To achieve the desired Trendelenburg position, simply pull the pin and tilt the bed in the desired directions. To bring the bed in its normal position simply pull on the pin and bring the bed back to its normal position.

3.5 Bed Movement Levers

The SMP-385MT is designed to achieve 3 movements (backrest adjustment, legrest adjustment, and height adjustment). These movements can be achieved by simply turning the designated levers located at the foot end of the bed.

3.6 Head/Foot Boards

The head and foot boards are designed to protect the patient from the environmental effects and increase the comfort of the patient. These boards are removable and can be installed easily. To remove the head/foot boards, simply pull on them by holding the top part and the boards will pop out of their holders. To install the head/foot boards, align the holes located at the bottom of the boards with the holders on the bed frame and simply push down to fix them in place.



USER MANUAL

MANUAL BED SMP 385MT

4. Bed Cleaning

All SMP manual bed models can be cleaned easily. The beds can be cleaned with a damp-dry duster. Make sure to discharge the patient out of the bed before commencing cleaning of the bed.

WARNING

NEVER USE ACID, DISSOLVENT, DETERGENT, PHOSPHATE OR ANY CHEMICAL INCLUDING PHOSPHATE OR ANY OTHER CORROSIVE AGENTS.

5. Storage

For the long term storage of the bed, obey the following instructions:

- Reset all of the adjustments of the bed
- Be sure that all the wheels of the bed are on a level surface
- Do not place any other bed or a heavy subject on the bed

6. Maintenance Inspection Schedule

TECHNICIAN NAME	SIGNATURE	DATE	EXPLANATION



USER MANUAL

MANUAL BED SMP 385MT
